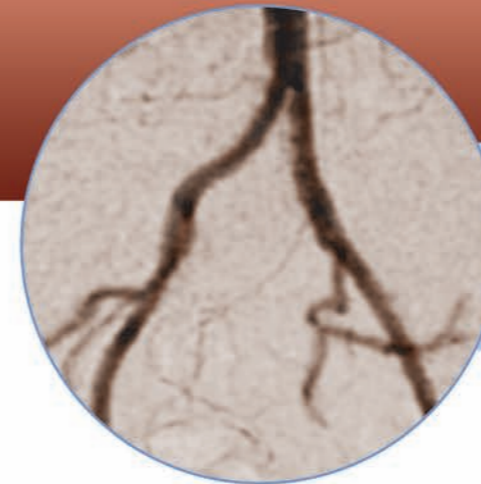
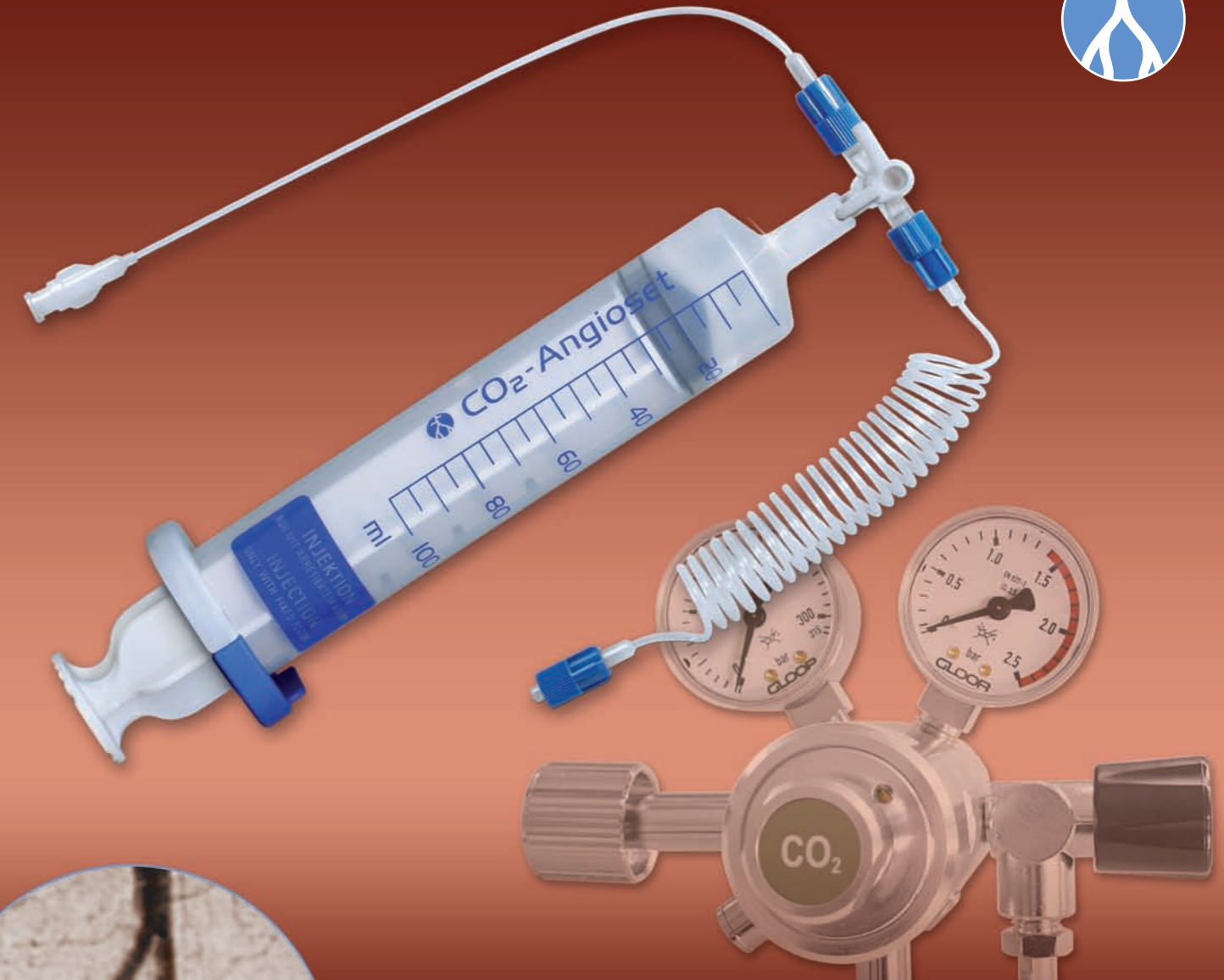
## CO<sub>2</sub>-Angioset

Bestell-Nr. Order code	Informationen Information
2100-0000	CO <sub>2</sub> -Angioset, Anschlüsse passen für alle OptiMed CO <sub>2</sub> -Druckreduzierventile CO <sub>2</sub> -Angioset, fits all connectors OptiMed CO <sub>2</sub> -Pressure Relief Valves

## CO<sub>2</sub>-Pressure Relief Valve

Land Country	Bestell-Nr. Order code	Informationen Information
Deutschland Österreich Schweiz	6676-530001010	Standard DIN 477-1
Europa universal	6202-35841002-EU	EU-Standard ISO 5145
France	6202-35221002-0	French-Standard AFNOR C
Great Britain	6202-35201002-0	British Standard BS 341, 3HW
	6202-35231002-0	British Standard BS 341, 3TF
USA	6202-35991002-US	US-Standard, Pin Index CGA 940
	6202-35851002-US	US-Standard, CGA 320
Australia	6202-35201002-AS	Australia Standard HW
	6202-35231002-AS	Australia Standard TF

# CO<sub>2</sub>-Angioset

Carbon dioxide angiography set

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*The alternative to conventional contrast media  
for an excellent image*

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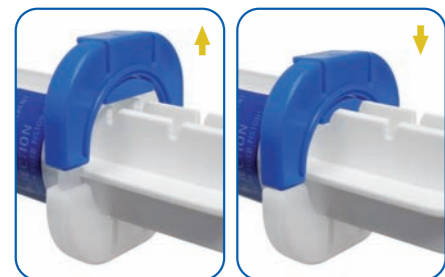
CO<sub>2</sub> as contrast medium

CO<sub>2</sub> is a safe alternative to iodine-based or low-osmolar contrast media. It is ideally suited for use in diagnostic and interventional radiology for patients with renal insufficiency, hyperthyroidism, or with an intolerance to contrast-media. Its use is as straight forward as with a conventional DSA.

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Indications

- Renal insufficiency
- Intolerance to iodized contrast medium
- Hyperthyroidism
- Presentation of pathological shunts
- Search for sources of haemorrhage
- Reduction of the loading with iodized contrast medium (e.g. in case of cardiac insufficiency or complex interventional surgery)

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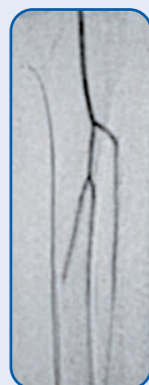
Applications

- Pelvic arteries
- Arteries of the leg
- Renal arteries
- Visceral arteries
- Haemodialysis shunts
- Venous displays (max. 40 ml per injection)
- Angiography of retrograde portal artery with TIPS

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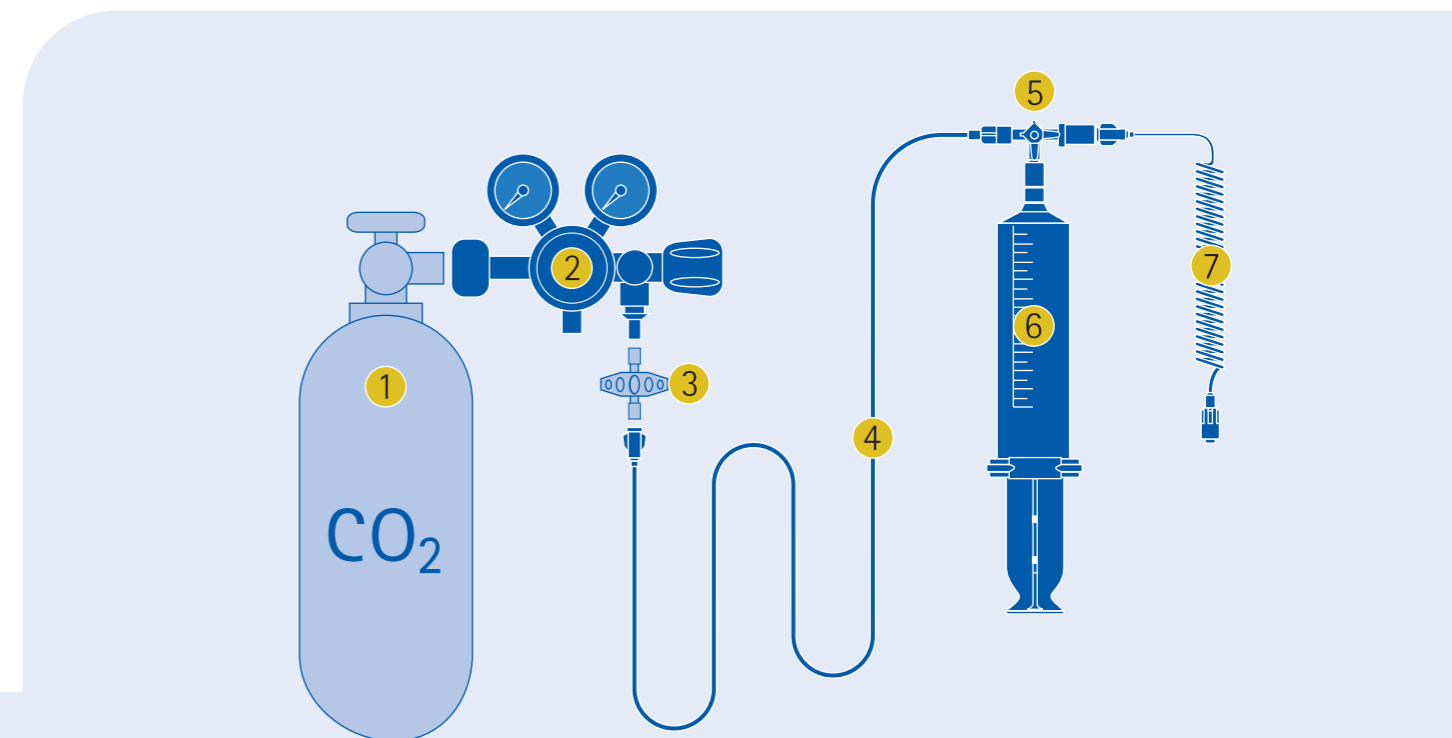
Safe pressure reduction

The CO<sub>2</sub> pressure relief valve is pre-set to 1.3 bar and enables a safe withdrawal of pure medical carbon dioxide from gas bottles. The two-stage pressure relief valve maintains a safe and constant pressure and features a safety and release valve.

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